


How to electronically save PDF
report of ABI test as one file

Download the free software called - PrimoPDF

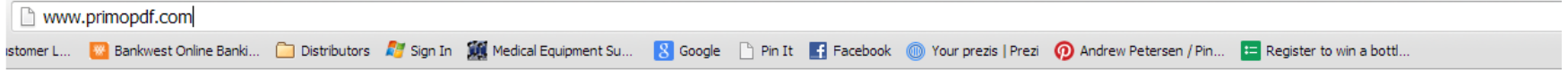

Our other free tools English



Downloaded 27,000,000+ times

Create PDF files with the world's most popular free PDF creator. With just one click, turn virtually any kind of file into a 100% industry-standard PDF


Try Primo



Download Free

Try Nitro Pro

Create PDF files, collaborate and review, fill and save forms, add text to pages, QuickSign documents, and more.

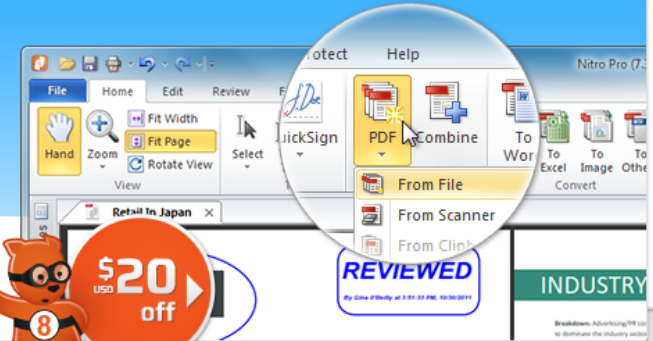

Download Free Trial




Features



Easy PDF creation


Convert almost any kind of file into a 100% industry-standard PDF file with one click.

Exclusive features for the desktop

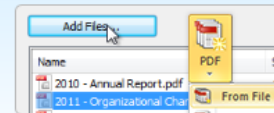


Batch convert & combine

Use Nitro Pro to batch convert large collections of files to PDF or PDF/A with just a few clicks. Combine files into polished PDF files, turn paper into fully searchable PDF files, and much more.

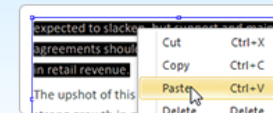

Edit and reuse content

When you'd prefer to directly edit a PDF or just export it back to Microsoft Word and Excel — Nitro Pro makes it dead simple to correct and repurpose your work.

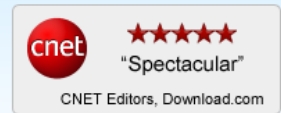

Software that's loved

Awarded the ultimate 5-star rating from CNET editors, the best PDF editor by Computerworld, and loved by experts and users alike.

Once downloaded

- open VitalVision Software
- Open report
- Go to print

The screenshot displays the VitalVision MS-2100 Software interface. At the top, the window title is "VitalVision MS-2100 Software" and the menu bar includes "File", "Patient", "Setting", "Print", "Other Tests", and "Help".

Patient Information:

ID	Name	Gender	Age	Height	Weight	BMI	Waist
	Jayden Sheridan	Male	22	187.0	63.0	18.0	76.0

ABI Results:

Right ABI	Pulse	Left ABI
1.10	57	0.94

Vital Signs:


Right Brachial	Left Brachial	Right Ankle	Left Ankle
Systolic: 124	Systolic: 114	Systolic: 137	Systolic: 116
Diastolic: 66	Diastolic: 66	Diastolic: 71	Diastolic: 68
Pulse Pressure: 58	Pulse Pressure: 48	Pulse Pressure: 66	Pulse Pressure: 48
ASI: 104	ASI: 61	ASI: 126	ASI: 77

The interface also features a central anatomical diagram of a human figure with colored markers indicating measurement sites: blue for the right brachial, green for the left brachial, and blue/green for the right and left ankles. A red arrow points from the "Go to print" step in the list to the "Print" button in the software interface.

Un-tick report 2 & 3 (as these are not necessary as they have the same information as report 1 & 4, but changed one graph to a trending graph, which is not needed).

Then go to “setting”

Select “PrimoPDF”

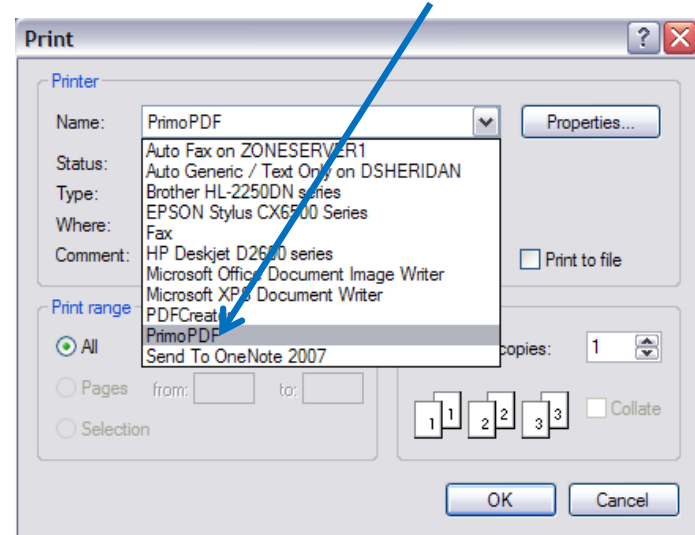

Then select “print”

Print Out

ID: 98723 Name: Jeff Abrahams Gender: Male Age: 61

VITAL VISION
Demonstration Zone Medical

Pulse Wave Analysis Report

Name	Mr. Jeff Abrahams	ID	98723										
Date / Time	2012/08/01 14:34	Gender	Male	Age	61yrs	BMI	32.3	Height	164.0	Weight	87.0	Waist	110.0

Measurement Result

R-Brachial ASI 101
L-Brachial ASI 73

PAD Test Result

Right ABI 1.12
Left ABI 1.03


Print

Post Process: Select “do nothing”

PrimoPDF window opens and click “create PDF”




Select the folder you wish to save the report, then Create name then save

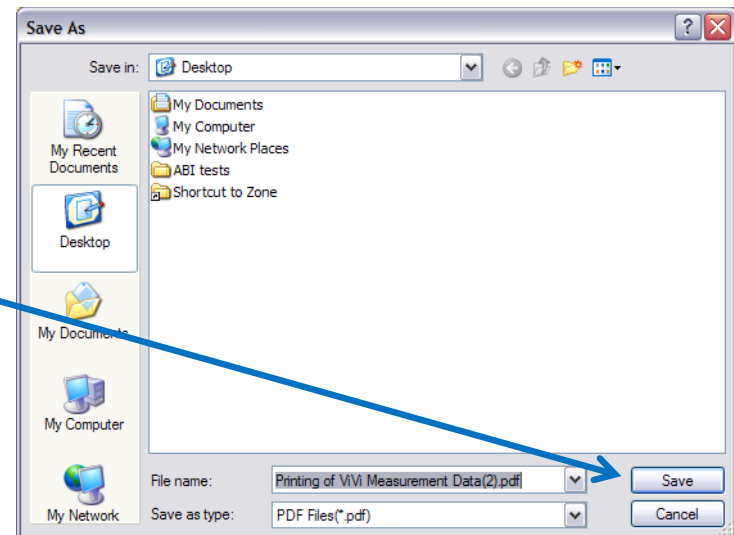


Then PrimoPDF window opens again to print second page




Click "create PDF"

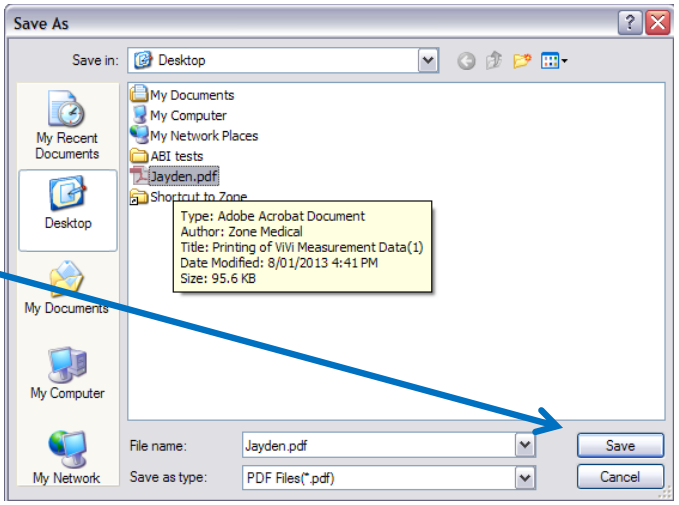

Choose same name, then save.

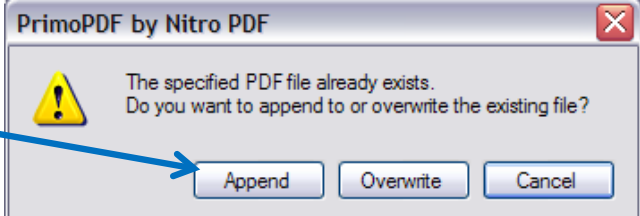

PrimoPDF will then give options. Choose "Append".

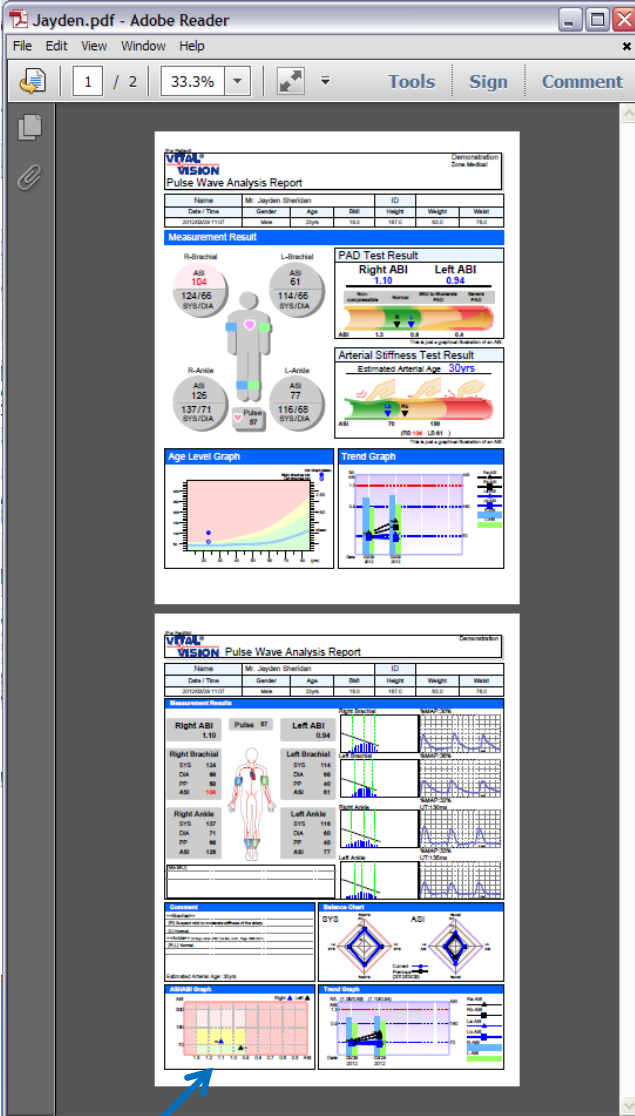

Both pages of report are now saved on one PDF file, which can now be inserted into electronic patient records.