



ASKINA® BARRIER FILM

LASTING SKIN PROTECTION FOR STOMA AND WOUND CARE

No sting, rapid drying, breathable skin barrier


Askina® Barrier Film

No sting, rapid drying, breathable skin barrier

When to use Askina® Barrier Film?



- 1 TO PREVENT** skin breakdown caused by **FRICTION** or **MOISTURE**
 - Protection of vulnerable areas: heels, elbows, toes, hips – from the first signs of reddening caused by friction.
 - Protection of incontinent patients' skin.
- 2 TO PROTECT** skin at risk from **MACERATION** and **EXCORIATION**
 - Periwound skin.
 - Peristomal area.
 - Around fixation sites for drainage tubes and external catheters.
- 3 TO MAINTAIN** the integrity of newly healed skin to stop secondary breakdown

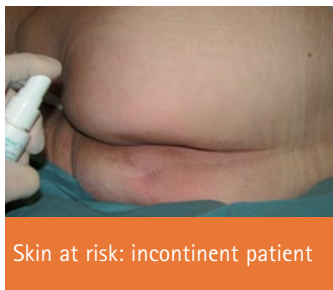

Skin at risk: incontinent patient



Incontinence dermatitis with skin breakdown



Peri-wound skin maceration




Tube insertion site

Indications Askina® Barrier Film

Askina® Barrier Film is indicated for use on intact or damaged skin. Askina® Barrier Film acts as a protective barrier:

- Against irritation from bodily fluids
- For sensitive and fragile skin
- Under adhesive dressings to reduce disruption to newly healing tissue
- For damaged skin
- For skin tears
- To protect periwound and peristomal areas
- For small cuts and tears
- At fixation sites for drainage tubes and external catheters

Completely sting free

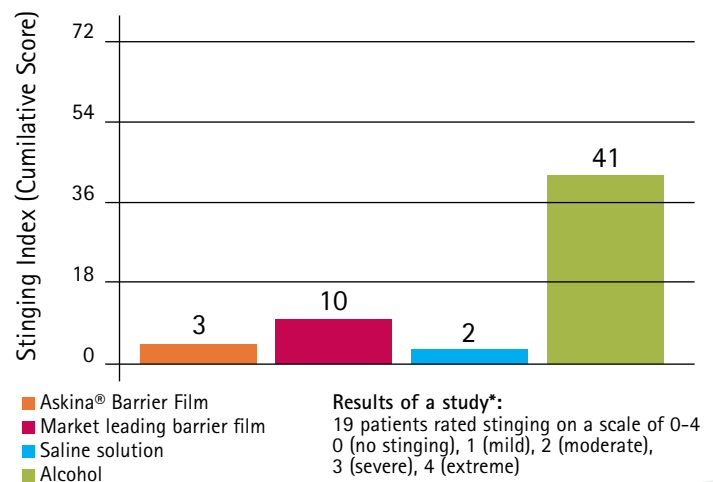

* Data on file

How to use Askina® Barrier Film?

- 1) Skin preparation**
 - Cleanse the skin with appropriate cleansing solution and dry thoroughly.
- 2) Application**
 - Holding the spray nozzle 10 – 15 cm from the skin, press the spray nozzle and apply a smooth coating of film over the necessary area.
 - Allows the film to dry for 30 seconds.
- 3) Removal**
 - If necessary, can be removed with any common medical adhesive remover.



Benefits in stomacare

Significant irritation of peristomal skin at day 1.




Signs of irritation have been almost entirely resolved at day 15.



After 15 days of use of Askina® Barrier Film, a net improvement of the peristomal skin is noticed.

Patient's main benefit in stomacare
Less pain during pouch removal

User opinion on skin protection efficacy¹

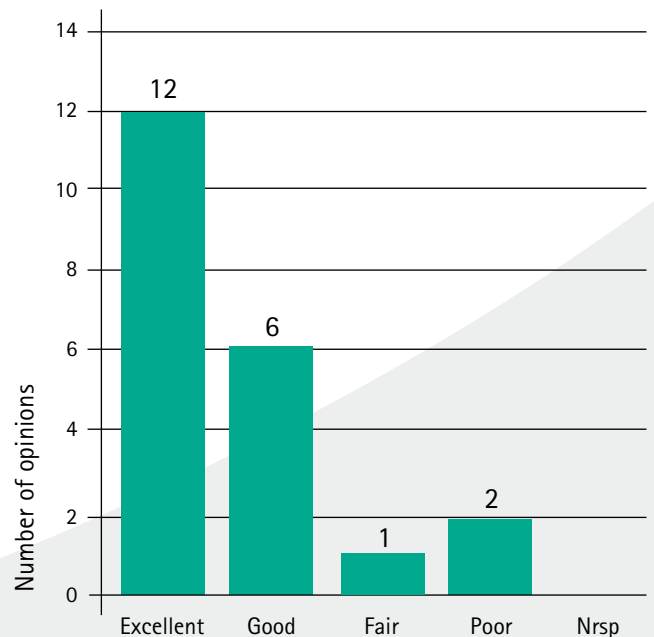

¹ 21 patients took part in the survey*

In summary, 18 out of 21 patients rated the product as «Excellent» or «Good».

Ordering information

Article number	Size	Units/Box
4002801	28 ml Spray Bottle	1
4002812	28 ml Spray Bottle	12

Manufactured by:

B. Braun Hospicare Lt. | Collooney | Co. Sligo | IRELAND
www.bbraun.com/wound-management

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and/or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun. Non-binding documents and photographs. For healthcare professional use only.