



# Zetuvit® Plus - Instructions for Use

## Exudate Management Pad

### Product Description

Zetuvit® Plus is a high capacity exudate management dressing pad with a highly absorbent central core to take up and bind exudate inside the dressing.

### Properties

The central components of the pad are an absorbent core of cellulose mixed with Super Absorbent Polymers (SAP) and a hydrophobic layer made from nonwoven polypropylene. These layers are encased in a nonwoven polyamide/viscose material.

### Mode of Action

The soft hydrophobic outer surface of the nonwoven polyamide helps prevent it from adhering to the wound, while the hydrophilic cellulose tissue of the inner surface creates a capillary action, which rapidly disperses exudate into the central core. The cellulose and superabsorbent polyacrylate (SAP) rapidly absorb exudate and reliably bind it. Exudate management assists with the removal of inhibitory factors from the wound.

### Indications

Suitable for wound bed preparation through exudate removal in heavy to severely exuding wounds. E.g. ulcers of various aetiologies, fungating tumours.

### Contraindications

- No known contraindications

### Precautions

- For external use only

### General Instructions

- Store below 40 °C
- Contents are sterile unless peel package is damaged
- Do not resterilise

### Application



1. Open peel pack and remove the dressing. Place white side down onto the wound – green side indicates upper side.
2. Secure with a retention dressing.

### Dressing Size

- Dressing should be 1- 2cm larger than the wound
- Cannot be cut to size

### Wear Time

- According to the amount of exudate
- During the early stages of wound management, the dressing should be inspected frequently
- Clinical protocols should be taken into consideration

### Dressing Removal

- To change the dressing, detach Zetuvit® Plus slightly at one edge and remove carefully

### Available Sizes

| Product       | Code    | Size       | Unit   |
|---------------|---------|------------|--------|
| Zetuvit® Plus | 413 710 | 10 x 10cm  | 10 Pcs |
|               | 413 711 | 10 x 20 cm | 10 Pcs |
|               | 413 712 | 15 x 20cm  | 10 Pcs |
|               | 413 713 | 20 x 25cm  | 10 Pcs |
|               | 413 714 | 20 x 40cm  | 5 Pcs  |

**PAUL HARTMANN AG – 89522 Heidenheim, Germany**  
**PAUL HARTMANN Pty Ltd – Homebush NSW 2140**



0 1 2 3



Attention:  
Refer to  
instructions



Do not  
reuse



Latex free



Store in a  
dry  
place



Temperature  
limitation  
10-40°C



Sterilised  
using dry  
heat. Do not  
resterilise



Cannot  
be cut